


## Digital Clinical Thermometer

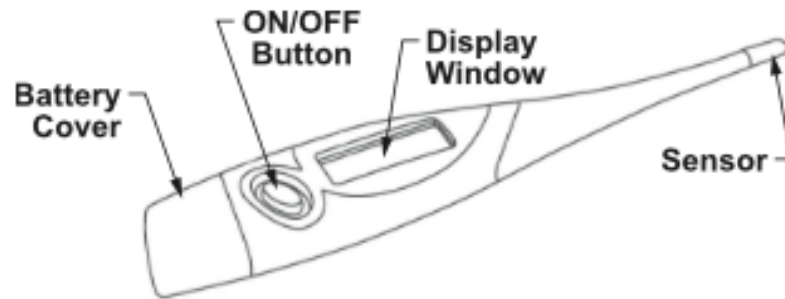

Model KD-135

Thank you for purchasing your brand new CVS Digital Clinical Thermometer. Please read these instructions carefully before use to ensure an accurate temperature reading.

### Parts Description



### How to Use

1. Always disinfect before use
2. Push the ON/OFF Button and the display will read  or  .
3. Release the ON/OFF Button and the display will show L°F with °F flashing.
4. Place the sensor in an appropriate position. Please see *Reliable Methods to Measure Temperature*.
5. Once the degree shows °F on the display, has stopped flashing and has beeped, the correct temperature reading will be shown.

### Reliable Methods to Measure Temperature

**Oral**-Place the sensor under the person's tongue. Instruct the person to close their mouth while the temperature is being measured. A normal oral temperature is usually considered to be between 96.8°F – 98.6°F

**Axillary**-Wipe the armpit with a dry towel. Place sensor under the patient's armpit and keep the patient's arm pressed firmly against their side. Axillary temperature is usually approximately 1.0°F – 2.0°F lower than oral temperature. *Note: In order to achieve better axillary temperature measurement result, a minimum measuring time of 3-5 minutes is recommended regardless of beeping sound.*

**Rectal**-If rectal temperature is recommended by your doctor, you may use a sensor cover (*purchase separately*). Insert thermometer tip into the sensor cover. Lubricate the sensor cover with a water soluble jelly for easier insertion. Do not use petroleum jelly. Insert tip of sensor no more than 1/2 inch into rectum. Stop if you meet any resistance. Rectal temperature is usually approximately 1.0°F – 2.0°F higher than oral temperature.

### Care Instructions

To clean the thermometer, wash the sensor with a solution of mild detergent and cool water. To disinfect the thermometer, wipe the sensor and lower stem with a clean cloth dipped in isopropyl alcohol. Do not boil thermometer.


Please keep out of reach of unsupervised children.

Keep thermometer at room temperature and out of direct sunlight.

Do not bend or bite the probe.

### Battery Replacement

1. When the symbol "⏏" appears on the display the battery needs to be replaced.
2. Pull the battery cover off in the illustration shown.
3. Use a toothpick to softly pry the battery loose. Avoid using any sharp tool for this procedure.
4. Place a new 1.55 VDC button sized battery type 192, LR 41 or equivalent in the chamber with the positive side up and the negative side down.

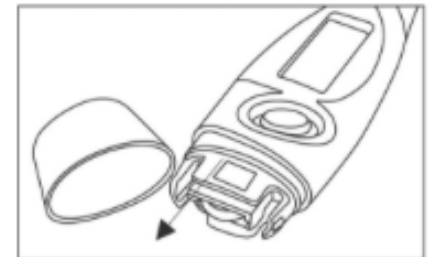

### Limited Warranty

All CVS products are satisfaction guaranteed or your money back!

Distributed by: CVS Pharmacy, Inc.  
Woonsocket, RI 02895

@2006 CVS/Pharmacy

Manufactured by:  
K-Jump Health Co., Ltd.  
Made in China